

How to take an antigen nose sample for COVID-19 testing before returning to the United Kingdom.

STEP 1 Wash your hands and place all items on a clean surface. Do not open more than one test pack at a time.

Do not dispose of any packaging.

STEP 2 Open the test and place on a clean, dry surface.

STEP 3 Read the instructions for use provided in the test kit.

STEP 4 Complete the test following the self-collection instructions. Label each test with your FULL NAME.

STEP 5 Wait 15 Minutes.

STEP 6 Place the completed test next to your passport open on the picture page.

STEP 7 Take a clear picture showing your test result and passport ensuring all details are clear.

STEP 8 Submit your details and upload the picture on our website.

<https://www.biograddiagnostic.co.uk/lateral-flow-data-collection>



STEP 9 Results and certificate will be sent within 24 hours to the email address provided.

Additional information

If you have any questions or problems, contact us by:

Telephone: 0345 565 1725
(08:00-19:00)

E-mail: clinical@biograd.co.uk

Data sharing

BioGrad diagnostics is legally required to share your data with Government Public Health Agencies, if requested the Police Service.

By returning your sample you are agreeing for your data to be shared.

Use by date and Storage requirements

Test kits must be used within 3 months of purchase and stored at room temperature & out of sunlight.