

How to take a nasal sample for COVID-19 antigen testing.

STEP 1 Wash your hands and place all items on a clean surface. Do not open more than one test pack at a time.
Do not dispose of any packaging.

STEP 2 Open the test kit and place on a clean, dry surface.

STEP 3 Read the instructions for use provided in the test kit.

STEP 4 Complete the test following the self-collection instructions. Fill out the antigen identification card with your reference number.

STEP 5 Wait 15 Minutes.

STEP 6 Take a clear photograph showing your test device, antigen identification card and passport ensuring all details are clearly visible.

STEP 7 Submit your details and upload the picture on our website,

<https://www.biograddiagnosics.co.uk/lateral-flow-data-collection>



STEP 8 Results and certificate will be sent within 24 hours to the email address you have provided.

Additional information

If you have any questions or problems, contact us by:

Telephone: 0345 565 1725

E-mail: clinical@biograd.co.uk

Data sharing

BioGrad diagnostics is legally required to share your data with Government Public Health Agencies and if requested the Police Service.

By submitting your sample details you are agreeing for your data to be shared.

Use by date and Storage requirements

Test kits must be used within 3 months of purchase and stored at room temperature and out of sunlight.